PUREBEAU  
FIBRO   
BLAST

2019/20

**Lifting without surgery  
The Original Made in  
Germany**

# Purebeau Jigsaw one fits to another

PUREBEAU COMPLETE BEAUTY  
PUREBEAU JIGSAW  
PERMANENT MAKE-UP · NEEDLING · FIBROBLAST · O<sub>2</sub>

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1000: Elke / Active 2017 Frankfurt 1410: Elke / Active 2017 Berlin

# Seminar itenary

- Introduction Purebeau Beauty Consultant
- Description of surgery
- Theory Fibroblast
- Preparation & Materials used
- Treatment plan
- After Care Management HydroMask / HydroPad und ColdBlast
- FAQ
- Practice
- Problem Management

# Purebeau Fibroblast - Lifting

- No injectable aesthetic required
- No cutting of skin
- No stitches required
- Very low risk procedure (NO risk of blindness as there is with surgery)
- Minimal side effects and far less downtime
- Quick & easy procedure
- Most patients can return to work the same day
- No thinning of the skin
- Significant cost saving

Become Purebeau Personal Beauty Coach – more than only an artist or esthetician !

A handwritten signature in blue ink, appearing to read "Henderson", is located in the bottom right corner of the page.

# Purebeau Catalog

22

**PUREBEAU FIBROBLAST**

**Lifting without surgery**

Forehead wrinkles

Frown lines

Lifting eyebrows

Upper lid tightening

Crowsfeet

Lower eyelid tightening

Nose wrinkles

Ear wrinkles

Nasolabial wrinkles

Mouth wrinkles

Chin wrinkles

Double Chin

Neck wrinkles

FIBRO BLAST top

Plasma flash

Epidermis

reduction effect

23

**A FIBROBLAST Control Unit with Pen**

FIBROBLAST is revolutionizing its holistic approach to the natural perfection of beauty with a new breakthrough method, the PUREBEAU FIBROBLAST. This allows the smoothing and tightening of skin without surgery. Device and pen including training.

Housing Colour:

Power: 50-80 Hz

**B THUNDER**  
Plasma Pen with plug-in power supply

**C FIBROBLAST Pens**  
Pens for PUREBEAU Control Units or power supply, including training

- 1x Thunder Pen with plug-in power supply
- 1x Thunder Pen for plug-in power supply
- 1x Thunder Pen with USB power supply
- 1x Thunder Pen with USB power supply

**D FIBROBLAST Tips**  
Disposable Plasma tips

- 1x Thunder Pen with plug-in power supply
- 1x Thunder Pen for plug-in power supply
- 1x Thunder Pen with USB power supply
- 1x Thunder Pen with USB power supply

**E EYE BAG BOOSTER**  
Disposable Plasma tips

- 1x Thunder Pen with plug-in power supply
- 1x Thunder Pen for plug-in power supply
- 1x Thunder Pen with USB power supply
- 1x Thunder Pen with USB power supply

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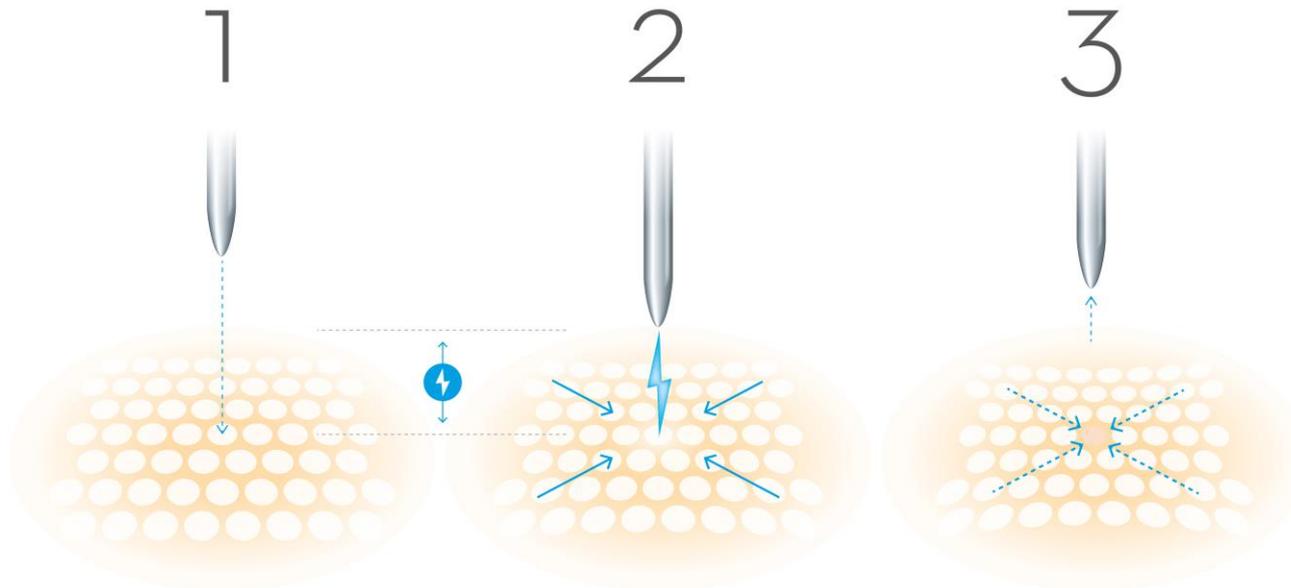
**G FIBROBLAST Powder**  
After Care Mineral Powder, bright, beige, tan

# Fibroblast Gear

# How does Fibroblast work

In the fibroblast handpiece, the electrical energy is converted into an electrostatic and transmitted to the applicator tip by impulse. At the right distance from the skin, oxygen and nitrogen are mixed, and the electrostatic energy produces the plasmatic effusion, which discharges on the skin with thermals. The skin pores open, the superfluous plasma or water evaporates, and with proper fibroblast technique we reach the contraction of the skin, whereby the excess skin dies down superficially and falls in the form of dandruff within the framework of the aftercare. The tightening is generated. At no time does the plasma injection penetrate the skin or cause an internal injury. The treatment is non-invasive. No medical practitioner or medical care is required.

PUREBEAU  
FIBRO  
BLAST

## Lifting without surgery

Example for an  
Upper Lid Tightening  
treatment

# When can we do it and when can't we do it

Patchtest

Feel the face structure and skin quality

See how skin react in which direction

Realize skin borders and determine how to work

Which technic is Better for which area

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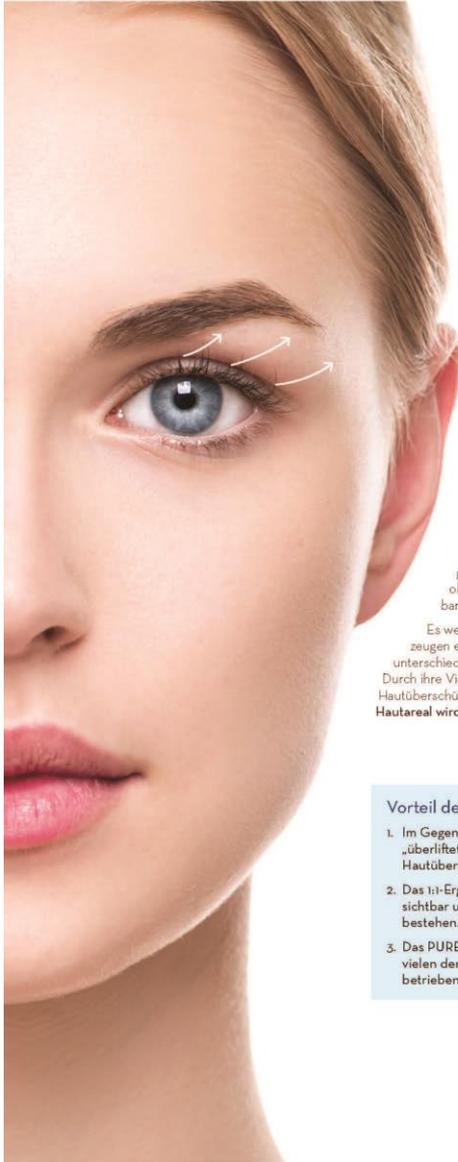
# Before Treatment

Careful Discussions regarding the clients reasons for wanting treatment are extremely important before beginning. You must also make sure that this treatment can deliver the results that they are seeking to achieve.

A medical history must be taken to make sure that there are no reasons why the clients should not undergo treatment

They must sign a consent form to say that they have understood the benefits and risks

# Consultation

This is an opportunity to learn about the clients history and any possible contraindications

## What should a thorough consultation include?

- Full details of skin type and history
- Lifestyle
- Ensure the client has realistic expectations
- Allow them to answer questions
- Explain the treatment
- Explain the aftercare and downtime
- Explain the frequency of treatments
- Treatment cost
- Ensure the client is not contra-indicated

# Fitzpatrick Skin Typing

**TABLE A2: ENVIRONMENT CANADA SKIN TYPES AND REACTION TO THE SUN**

<b><u>SKIN TYPE</u></b>	<b><u>SKIN COLOR</u></b>	<b><u>HISTORY OF TANNING &amp; SUNBURNING</u></b>
I	White	Always burns easily, never tans
II	White	Always burns easily, tans minimally
III	Light Brown	Burns moderately, tans gradually
IV	Moderate Brown	Burns minimally, tans well
V	Dark Brown	Burns rarely, tans profusely
VI	Black	Never burns, deep pigmentation

# Treatment Procedure

- Cleanse treatment area
- Gather photo evidence
- Apply pre cream
- Remove section of cream and ensure area is dry
- Perform one shot test to check client's sensitivity
- Carry out 1 area sit client up take photo (optional for better 'after' photos)
- Follow on with other side
- Apply cold compress
- Apply tinted make up and SPF
- Explain aftercare

# Treatment plan

- Cleaning with Octenisept & compresses, no other cause inflammable reactions
- Numbing ca 40 – 45 pre , with clear fresh tape airtightened – after spray you may numb again Take off of EMLA area wise and start fibroblasting

Working time per area is approx 10 - 15 min, start numbing area 1 and 10 – 15 min later working time 2 to avoid loss of numbing. Clean with Octenisept and cold bit wet kompresses out of cooler. For first day use Tyrosur Powder or Gel, Lavendel Oil or Bio Cocos fat if needed after HydroMask or HydroPad. And ColdBlast, which still works for fast rejuvination and kills bacteria. After that use the next days always the after care with SPF and cool with cold compress. After care has to be taken until the Skin is completely healed. There will be some swelling for another 3-4 days on the treatment area. A lymph drainage might help. After Care Mineral Powder will cover the area and shelter it as well. Take SPF and after Care as often you feel and as long as the healing is not finished. Don t work too long and too hard on the skin, to avoid a hyperpigmentation Or long healing periods. We recommand using Collodial silver liquid to clean the area and put one layer on before you use the Fibroblast After Care and the mineral powder to cover. This has to be done several times a day as you may feel dryness. Good versus swelling is Lyphmdiaral.

# Pre & After Care

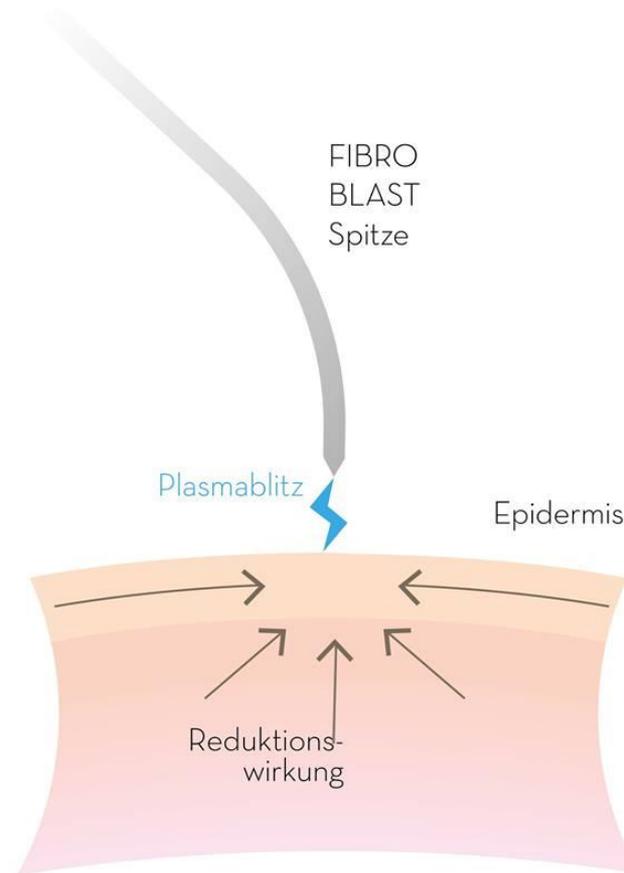
# After Care – first aid

Purebeau HydroMask & Pads & ColdBlast


# Function Fibroblast-**Spotting**



# Function Fibroblast-**Spotting**

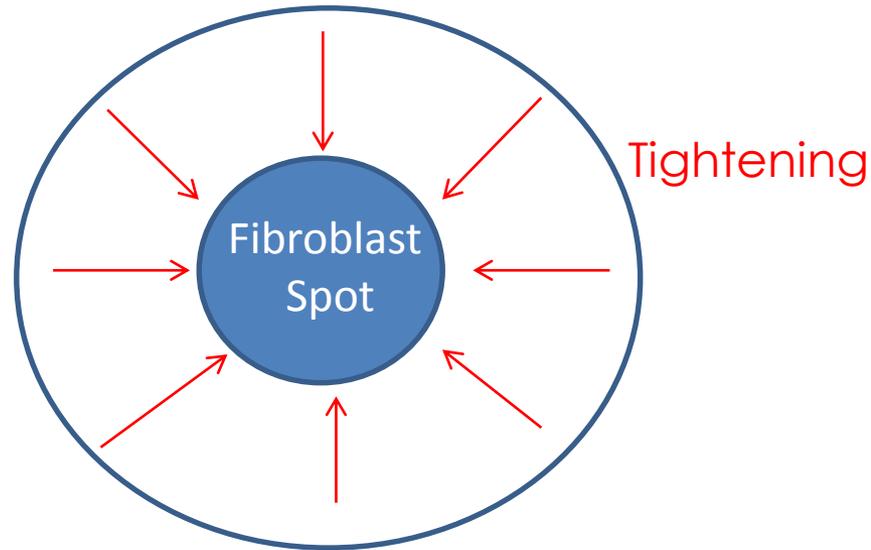Reaction radius


# Fibroblast Spotting

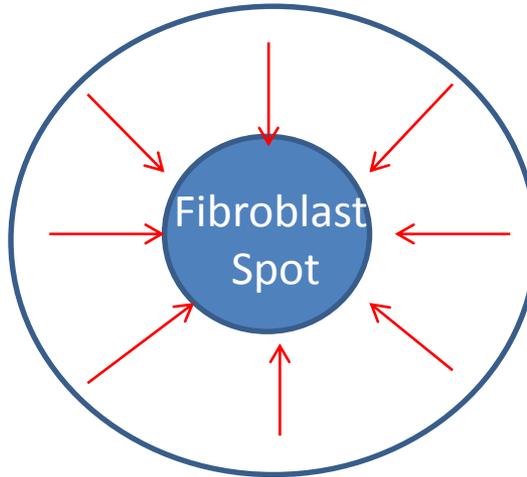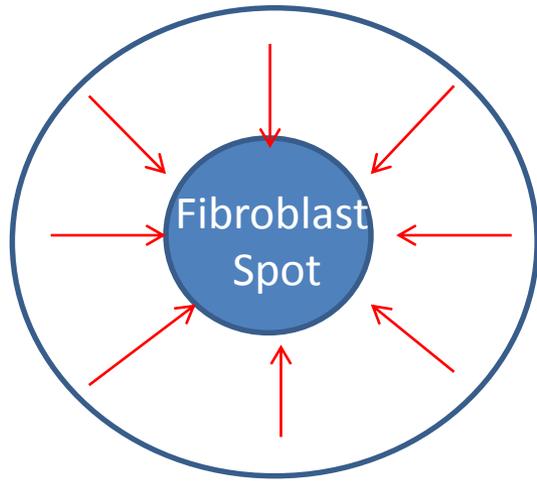
Leave enough  
space between  
To get the best  
lifting results count  
1,2,3  
And spot next

**DON'T:**


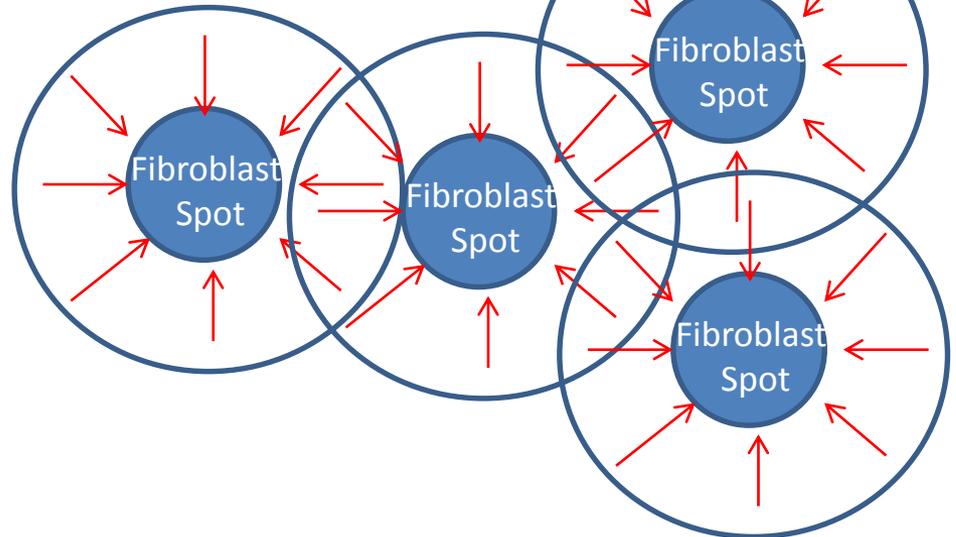
# Fibroblast Center Spotting

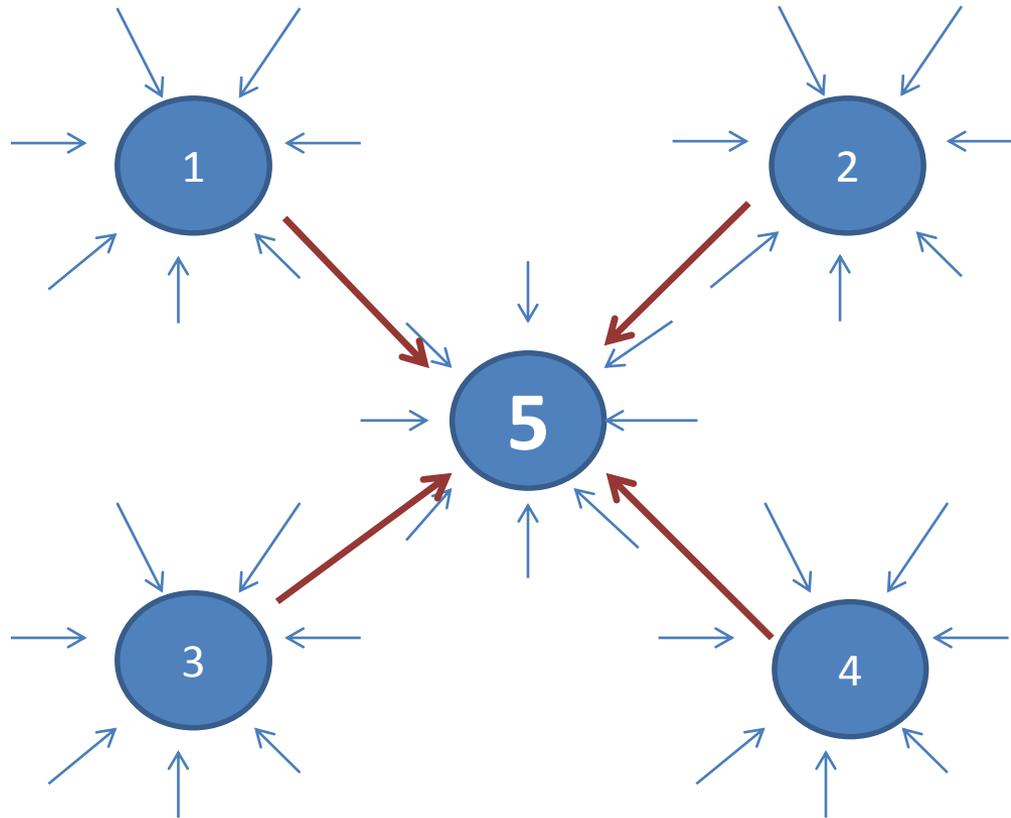
Example. Watch purebeau Fibroblast on Youtube or purebau


# purebeau Fibroblast Technic
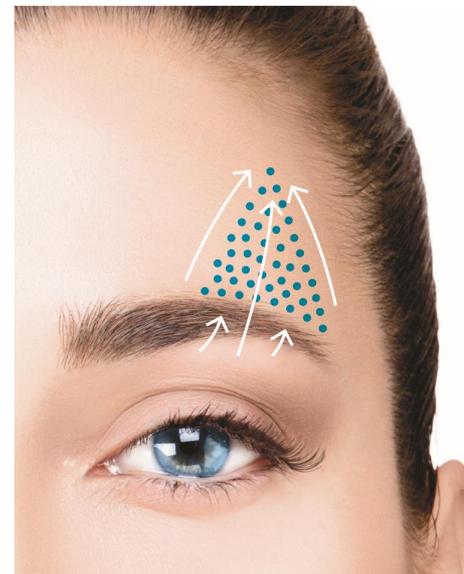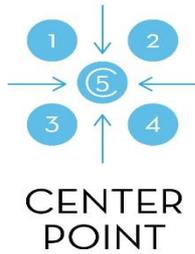
# Where do we start upper lid and how to work the wrinkle away

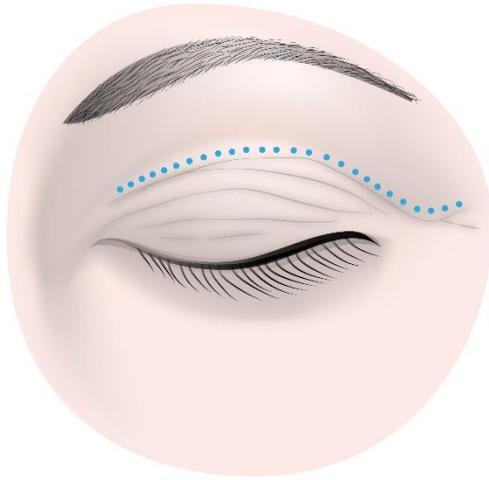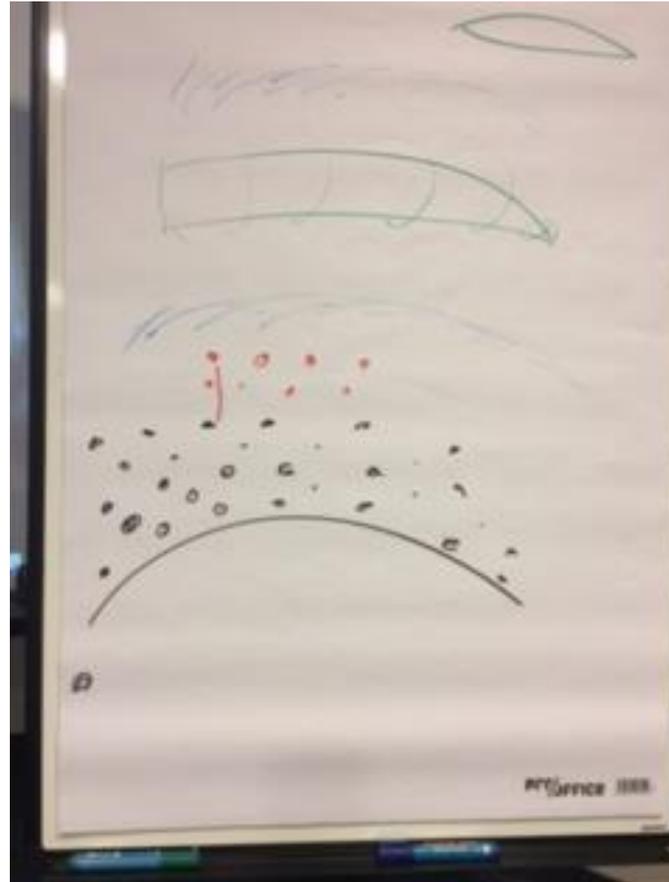
Fibroblast the highest or lowest point  
to optimize lifting



# Fill in Fibrotechnic

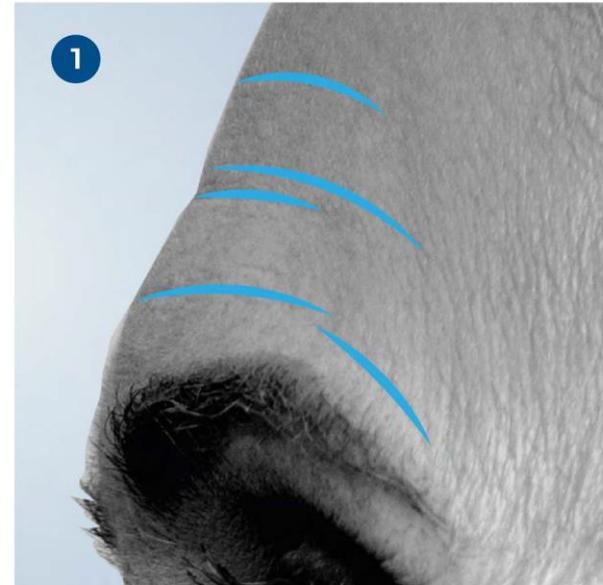
- |                     |                     |                    |                |
|---------------------|---------------------|--------------------|----------------|
| 1) Stirnfalten      | 4) Oberlidstraffung | 7) Nasenfalten     | 10) Kinnfalten |
| 2) Zornesfalten     | 5) Krähenfüße       | 8) Nasolabialfalte | 11) Doppelkinn |
| 3) Augenbraue heben | 6) Unterlid         | 9) Mundfalten      | 12) Halsfalten |


# Fill in

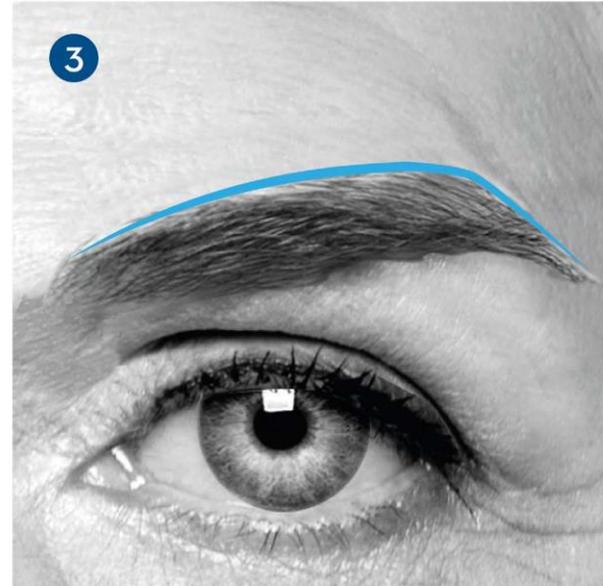
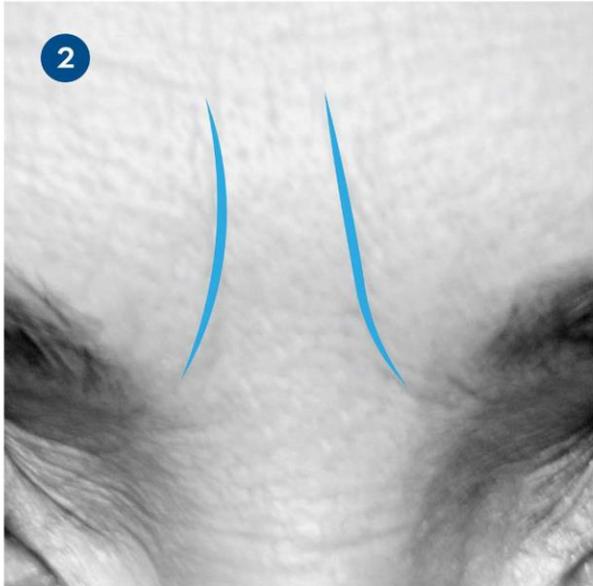
# Fill in


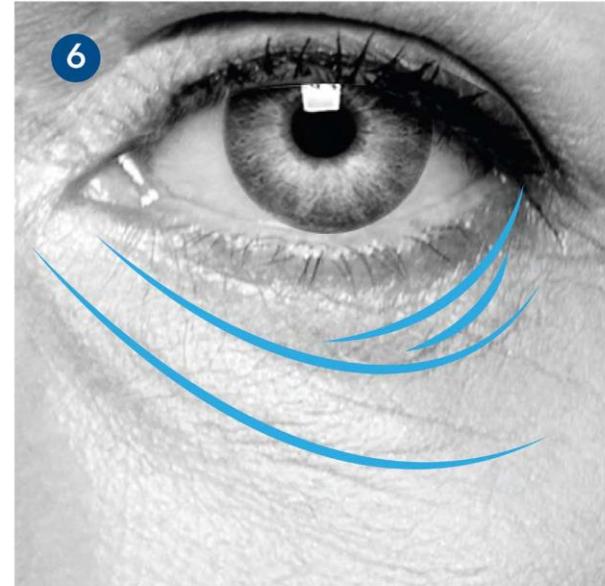
# Fill in
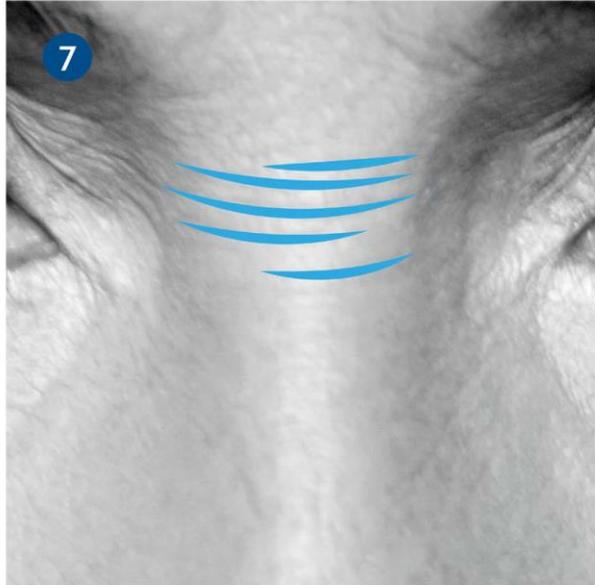
# Fill in
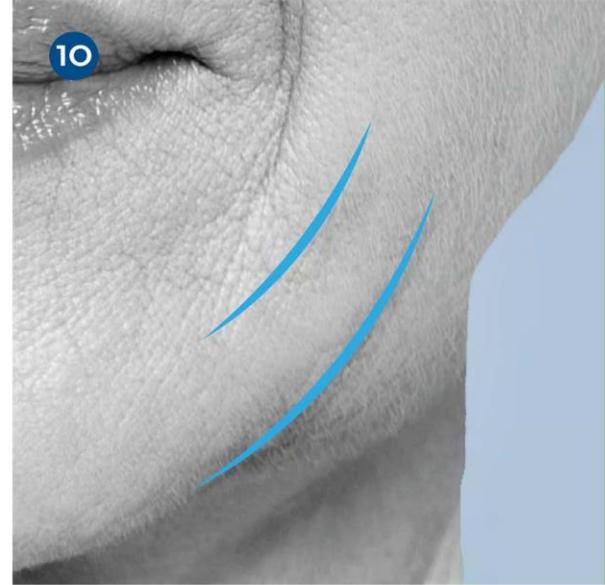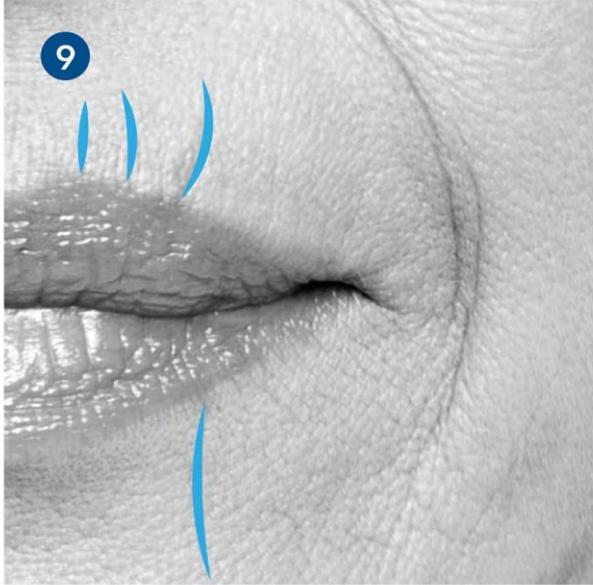
# Fill in
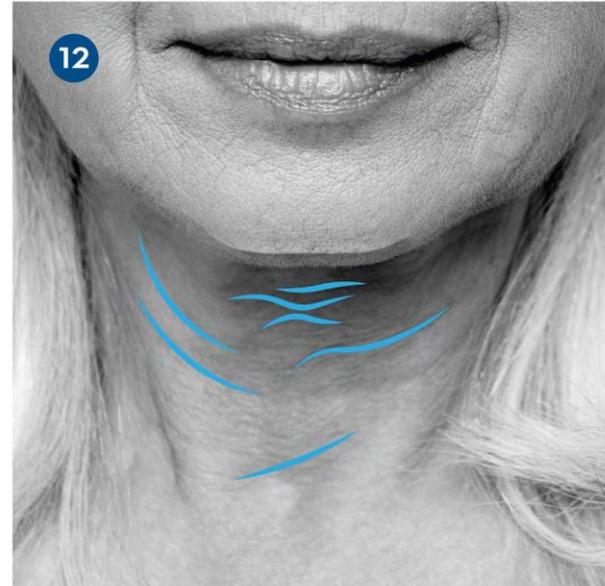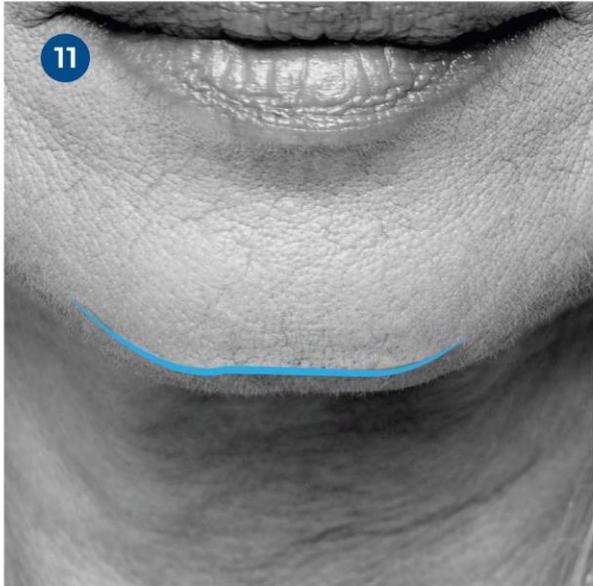
# Treatment book

PUREBEAU  
FIBRO  
BLAST

# Wrinkle technic

# Parts of work

**Area: Breast**

**Age: 62**

**Two days after one treatment**

# Parts of work

**Area: Cheek**

**Age: 56**

**One day after one treatment**

# Parts my work

**Area: Neck**

**Age: 60**

**Five days after one treatment**

# Parts of work

**Area: Neck**

**Age: 55**

**Two days after one treatment**

# Parts of work

  
Nathalie

**Area: Stomach**

**Age: 53**

**Six days after one treatment**

# Parts of work

  
Nathalie

**Area: Forehead**

**Age: 62**

**Four days after one treatment**

# Parts of work

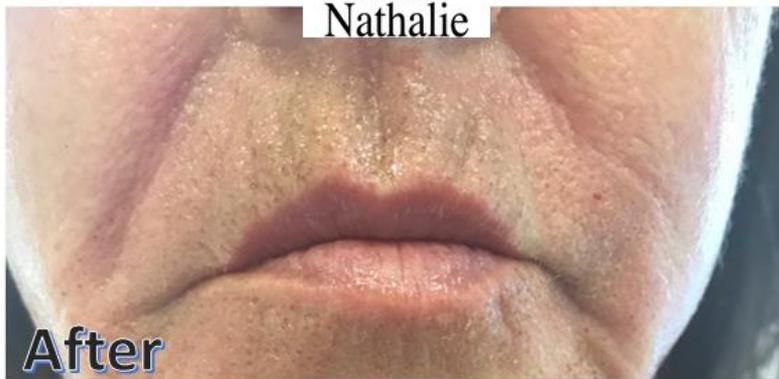
**Area: Eyes**

**Age: 54**

**Five days after one treatment**

# Parts of work

  
Nathalie

**Area: Upper Lip**

**Age: 62**

**An hour after one treatment**

# Parts of work

**Area: Cheek**

**Age: 56**

**A day after one treatment**

# Parts of work

**Area: Neck**

**Age: 49**

**Six days after one treatment**

# Fibroblast Treatments

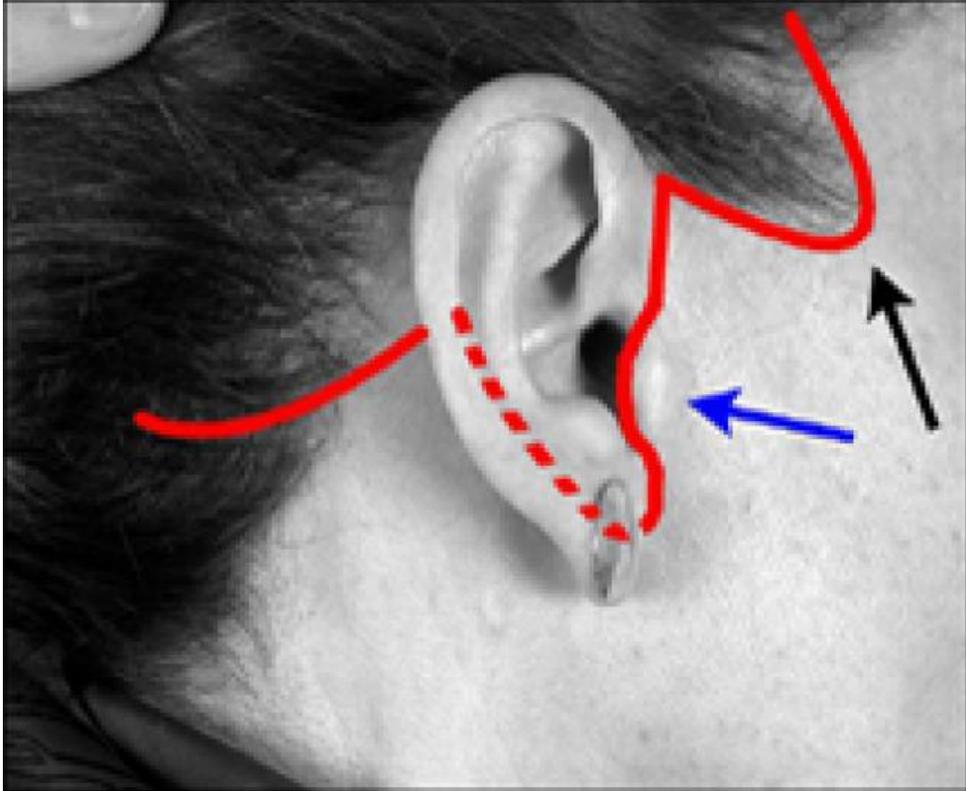
*Christina Rivera academy*

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- Apply the normal pre-treatment routine before carrying out the aesthetic treatment.
- Mini-facelift by using fibroblast is usually carried out by applying the skin tightening where the skin would be pulled during a normal face lift procedure, which is close to the ear as shown in the video above.
- It is important not to manage the client expectations of this type of procedure, because the improvements will only be minor and cannot be compared to the dramatic results achieved with surgery. However since, the localised skin tightening effects of electrical arcing are cumulative, repeating the treatments a number of times will lead to good results.

# Parts of work

**Area: Ear**

**Age: 58**

**A day after one treatment**

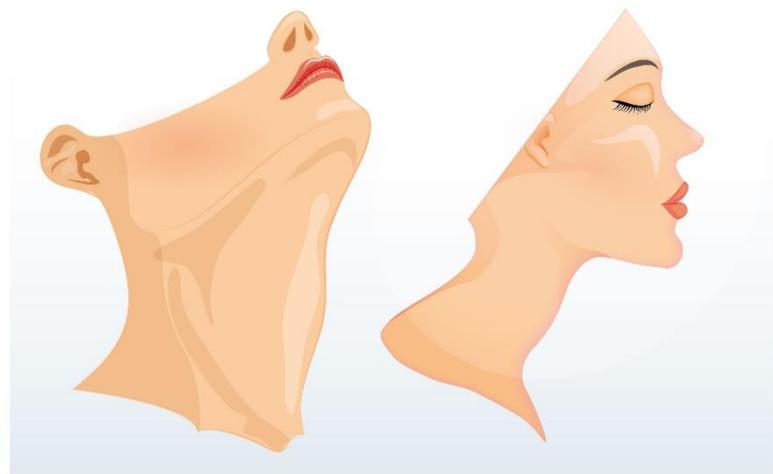
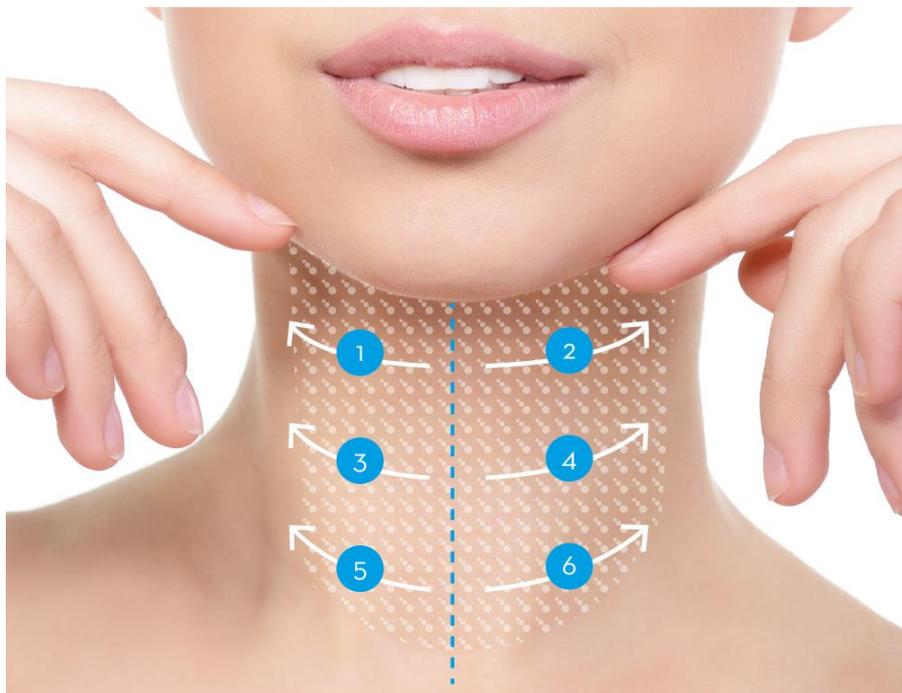
# Fibro Lifting Face

# Fibrolifting


# Spraytechnic


# Neck Lifting by Fibroblast Spraytechnic

PUREBEAU  
FIBRO  
BLAST

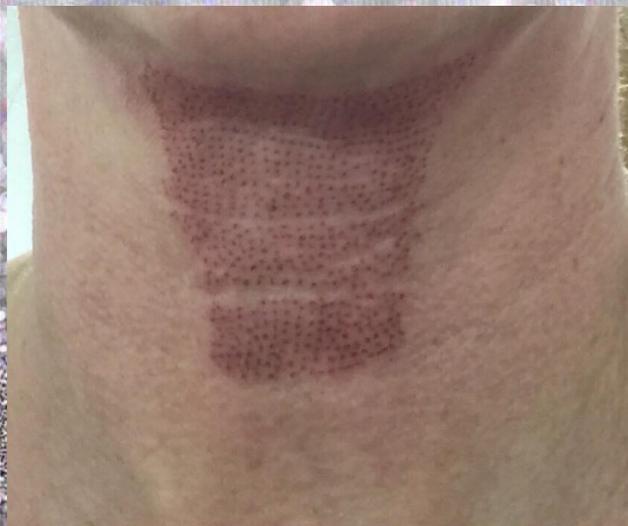
*Elle Marie  
Permanent Cosmetics*


# Combiblast

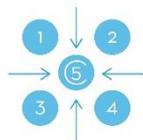
CENTER  
POINT

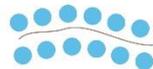FALTEN  
SPOTTING

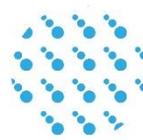
SPRAY  
SHADE

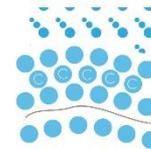
COMBI  
BLAST

# Fibroblast Body Check
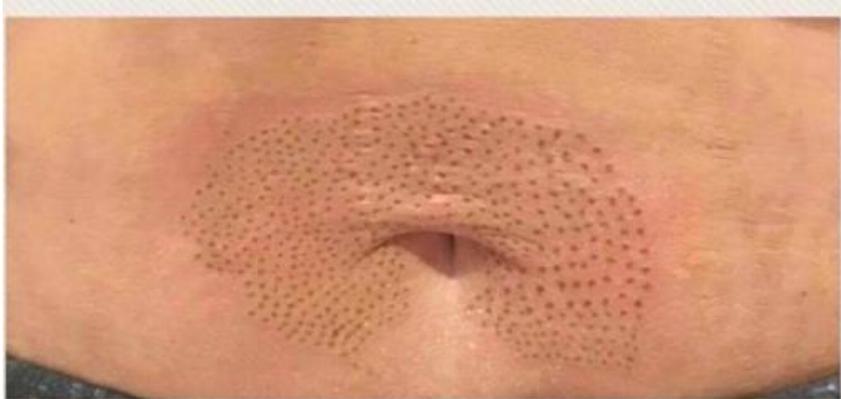
# Fibroblast Treatments

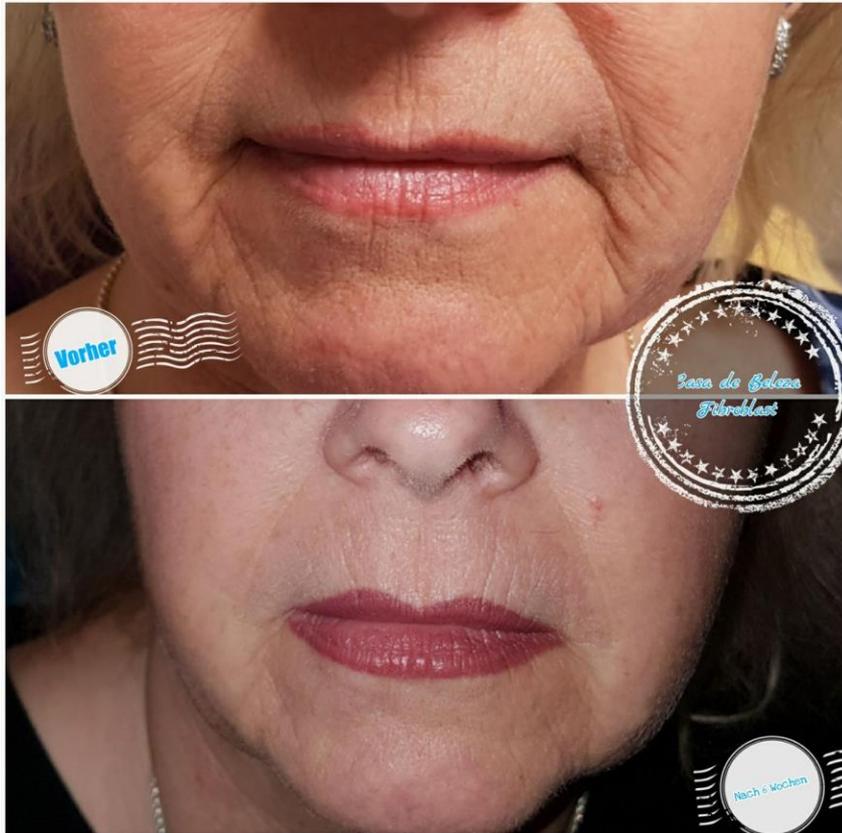
# Fibroblast Treatments

# Dragonblast

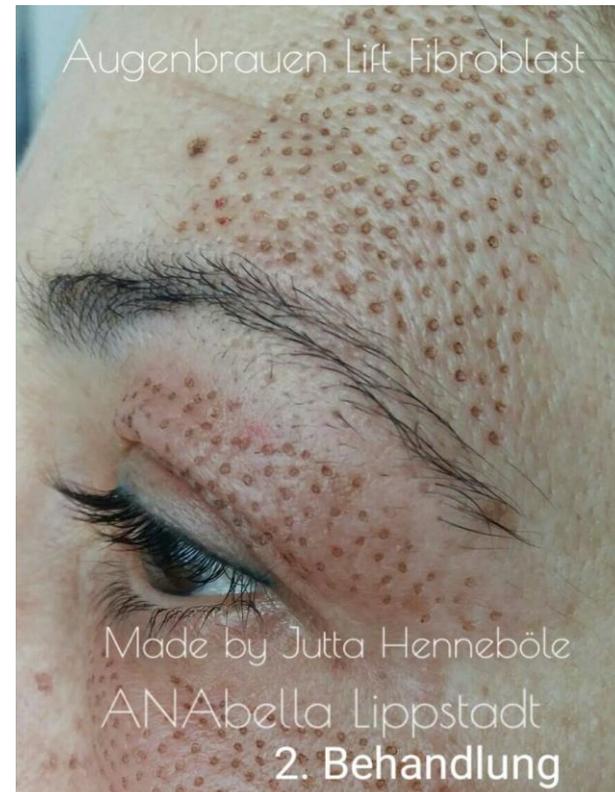- There are various areas which can be treated in various forms. To check for the right direction you can stretch with your fingers and play the skin flow.

# Removing Moles

- Mole checked first
- Spray Operation
- Start burning the top of the mole with the equipment
- Wipe off the carbon residues after each burst to appreciate the level you have reached.
- Repeat the process until you have almost reached the surrounding skin level.
- Every now and then the tip of the electrode will become clogged with carbon residues, at that point
- Soothing product and antiseptic immediately after the removal.

# Removing Syringoma

- Syringomas come in many different shapes and colours, they can be oval, dome shaped or round. Periorbital Syringomas can be both clustered unevenly in one region only or quite well distributed throughout the area of the skin around the eyes or the eyelids.
- Periorbital Syringoma are skin-coloured or yellowish firm rounded bumps, 1-3 mm in diameter.
- Apply the spray mode to simply levelling the Syringoma off with the surrounding skin as shown in the video above. Remember to wipe off the carbon residues from time to time, preferably after every spraying operation.

# Removing Skin Tags

- Apply the electric arc on top of the skin tag and work all the way down until you have **almost** reached the base of it (skin level).
- At that point if you have not already done so, reduce the power level of the device to the minimum to allow the greatest degree of precision.
- Continue to wipe the carbonisation off the skin tag until you have levelled it off with the surrounding skin. Remember to wipe off the carbon residues from time to time to allow you to see the level you have reached.

In case the skin tag you are removing is particularly large you can apply the arc onto an area of the skin tags itself close to skin level. Once the skin tag has become detached, you can then proceed to level off the remaining skin tag residues with the surrounding skin in order to accomplish good results

# The future of the fibroblast treatment

Starting to implement the vision

# Client consent form

Declaration of Consent for the  
PUREBEAU FIBROBLAST-Treatment

Seite 1 von 2

## 1) Declaration of Consent for the PUREBEAU FIBROBLAST treatment

yes  no I have been personally advised by the handler about the type, kind and purpose of the treatment, including information about possible anesthetization.

I was thoroughly informed about the required behavior, as well as the necessary sun protection before and after the treatment, and pointed out possible complications before and after the treatment.

In doing so, my personal situation was sufficiently discussed, as well as realistic treatment results. I have received, read and understood the leaflet with general information for the patient on treatment and after-treatment. I was also able to ask all the questions I was interested in. These were answered and understood by me i.e. Specific personal risk factors of the patient (medication, operations, sensitivity to light, especially disorders):

In order to be able to perform your FIBROBLAST treatment optimally, we ask you to answer the following questions:

yes  no Are you HIV positive?

yes  no Were you or will you be treated with cortison during the treatment?

yes  no Do you suffer from heart problems

yes  no Do you take medication on a regular basis?

yes  no Do you have a pacemaker?

yes  no Are you allergic, if yes against what?

yes  no Do you take blood thinning drugs?

yes  no Do you have a healing disorder?

yes  no Do you suffer from Herpes?

yes  no Infectious diseases, if yes, which disease?

yes  no I would like an allergy test (surface-anesthesia)

yes  no You are of age, and in full possession of your mental state and do not have a legal guardian and make your own responsible decisions.

Declaration of Consent for the  
PUREBEAU FIBROBLAST-Treatment

Seite 2 von 2

## 2) Areas of application with PUREBEAU FIBROBLAST

▷ Age-warts (seborrheic keratoses), Fibromas (predunculated wart), skin-colored nevi (dermal nevi), viral warts, age spots (lentiginos) and light damaged skin (actinic precanceroses)

▷ fat deposits under the skin of the eyelids (Xanthelasm)

▷ scars, acne scars

▷ wrinkle smoothing, eyelid lifting

## 3) Risks

Even if the therapy is carried out in the correct manner there are certain risks, which are listed below:

▷ Intolerance of the local anesthetic (cream form or local injection)

▷ Wound infection / wound healing disorders /scarring (extremely rare)

▷ Pigment disorder (hyper hypopigmentation) / a sunscreen with a high filter SPF 50 should be used for at least 2-4 weeks.

## 4) Declaration

I am aware that a guarantee cannot be given for the results of the treatment. I have also been informed about the necessity of additional treatments (fee required), which may be necessary to achieve the desired results.

Nevertheless, I agree to carry through with the above described treatment. I have been given sufficient time and opportunity to overthink my decision and I do not have any further questions, as all of my questions have been answered completely and thoroughly understood.

I have received and read the patient information. I will follow the instructions. I agree to the fibroblast lift treatment after sufficient time for consideration.

I agree to the procedure with the PUREBEAU FIBROBLAST immediately and waive a 24-hour consideration period.

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Stefan NAI  
Rechtsanwalt

# Law judgement & Lymphdrainage

ENGLISH TRANSLATION OF THE JURISDICTION OF FIBROBLAST  
FOR COSMETIC TREATMENT by Segelken & Suchofar advocates

We come back to the question if the purebeau Fibroblast treatment requires a license in accordance to § 1, pt.1 of the non-medical Practitioners Act, or if it is a cosmetic treatment.

According to your information the Fibroblast Pen converts a direct current into an electrostatic impulse, which is transmitted to the sterile disposable applicator, thus creating an oxygen – nitrogen mix and with the correct distance to the skin discharges a plasma flash. The oxygen-nitrogen molecules shrink the plasma, causing the skin to tighten leaving a soft crust of dead skin which slowly flakes off due to by the fast rejuvenation caused by oxygen and the renewal of fibroblast, i.e. cells which play a main roll in the renewal of the extracellular matrix. At no stage do we have an invasive impulse going deeper than the upper layer of dead skin.

The electrostatic impulse is not a laser or an IPL, non-invasive and is free of any contraindications operated by authorized and educated personal.

According to §1, part 1 of the non-medical Practitioners Act a person wanting to “heal” without being a doctor must have a certification as a natural health practitioner. This law defines in §1, pt. 2 the “art of healing”. Accordingly, one who is finding, healing or treating illnesses, ailments or bodily injury is a licensed practitioner. These subjects are well defined in this law. “Detection”, is determining if an illness is given. “Healing” is a conclusive elimination of a condition, which is beyond the norm. “Relieving” is to better a condition, which is beyond the norm.

The fibroblast treatment concentrates on the lifting of skin and wrinkles, caused by normal human aging. Under this aspect it is not a condition according to the Practitioners Act. The subject of illness has a wide aspect. Under this, it is definitely not the normal functional and performance fluctuations, which the body is subject to by nature, to be understood.

“Conditions” are long lasting and non-therapeutically repairable malfunctions, and “illnesses” are to be understood as malfunctions of body, organs or parts of organs.

Systematically speaking is that age-related wrinkles are not an illness, condition or bodily injury.

In reference to the judgment of the court in Lüneburg from July 20.2006 - 8 LC 184/04, is that the protective purpose reaches further than the preceding. §1, pt. 1 of the non-medical Practitioner Act targets the protection of the human condition.

Under consideration of this protective purpose should such treatments, which are carried out only for cosmetic or aesthetic reasons, fall under the required licensing- but with incorrect use cause significant bodily injury.

The professional Fibroblast treatment is not causing any contraindication or risk if carried out by a trained esthetician. Even an inappropriate treatment cannot cause severe damage. In accordance to this there is no approval necessary by the non-medical Practitioners Act.

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# Complications caused

1. Insufficient clarification of the customer
2. Incorrect fibroblast in length and shape
3. Failure to comply with the management plan
4. Deferred aftercare with other aftercare products
5. second treatment too early scheduled
6. Omitted problem management

# How hyperpigmentation is caused

Hyperpigmentation by hormones, see Melasma.

These pigment spots are almost always found in the face, almost only in women

Age spots, sun spots These hyperpigmentation can be found, for example,

On the back of the hand, and in the face, usually a few millimeters to centimeters large flat spots

Brown pigmentation after previous skin inflammation, eg neurodermatitis, psoriasis, lichen ruber ...

These pigment spots can occur all over the body, usually as a blurred dark coloration of larger skin areas

Hyperpigmentation after sun or UV rays (natural or excessive tanning): often after sunglasses as spotted tanning of the shoulders

Adverse effects in case of insufficient sun protection after laser treatment or peeling: as darkening of the lasered areas

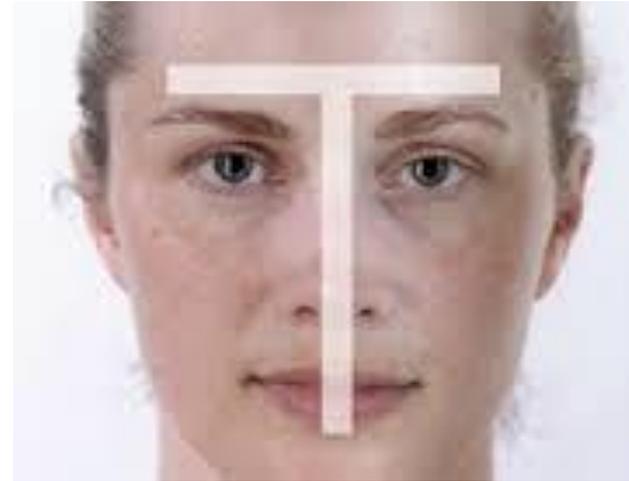
Hyperpigmentation after insect bites or other skin infections: often on the lower limbs with pea-shaped hardening of the skin

Internal disease (eg adrenal disease, tumor, malocclusion): usually as uniform brown coloration of the whole skin


Side effects of perfume / plant juice + sun: often on decoll tea, arms or face after spraying of perfume or gardening with plants and subsequent tanning

Light or dark spots on the upper body with fine scaling: caused by the brackish mushroom

# Hyperpigmentation

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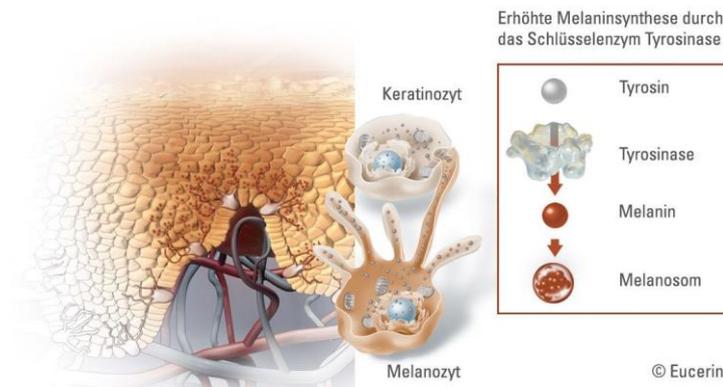# How to prevent Hyperpigmentation

- By working gentle
- By using our Fibroblast technique in the appropriate rhythm
- By using frequently the appropriate Fibroblast After Care
- By applying as add on cleaner and layer below the After Care from Fibroblast Colloidal silver

# Colloidal silver

- In the case of skin rashes, the skin can be rubbed with a paper towel impregnated with colloidal silver or impregnated cotton wool.
- When the eczema is closed, the skin can be rubbed with a paper towel impregnated with colloidal silver.
- In the case of open wounds, the wound area can be sprayed with colloidal silver in order to avoid pressure and material contact.
- In pressure sensitive injuries such as sunburn or light burns, the colloidal silver can be sprayed on.
- For pimples, colloidal silver can be used like a facial water.
- In the case of furuncles or badly healing wounds, a patch or dressing with a compress impregnated in colloidal silver may be applied.
- Where to buy: Amazon / pharmacies / concentration 25 ppm – 40 ppm

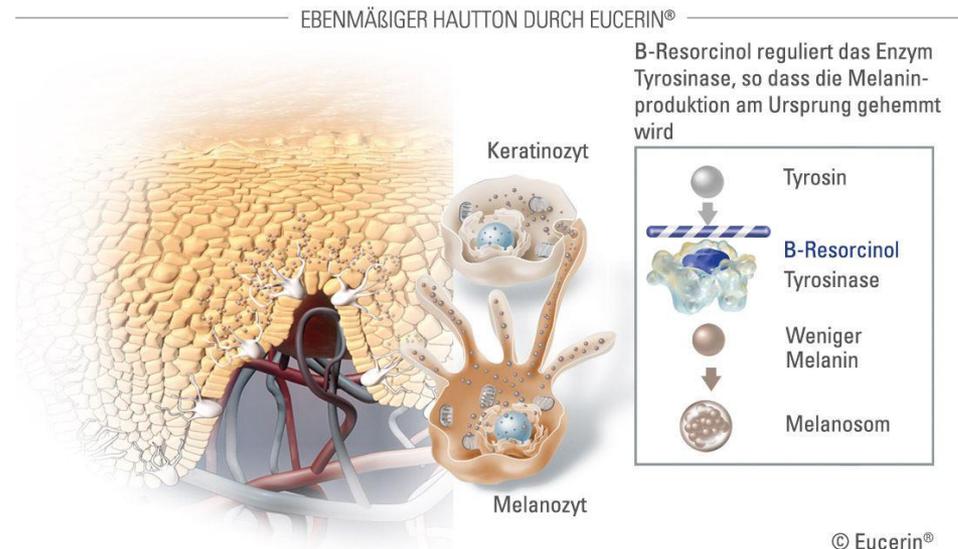
# How to remove hyperpigmentation

Removal - laser treatment, high-energy light pulses or chemical peeling destroy the excessively pigmented skin cells and reveal new skin without discoloration.

However, these methods can be costly and invasive and can degrade the skin image. Potential side effects include inflammation, irritation, and burning.

Eucerin Even Brighter, which contains B-resorcinol, has been shown to lighten dark spots within four weeks. Skin irritation rarely occurs.

# Attention

Affected persons should note in any case that: The UV rays in the sunlight are a great influence factor of the skin. Sun exposure will probably intensify the symptoms. So, not only the sun should be avoided, but it is also recommended to apply a sunscreen with a high SPF and a broad spectrum of effects within the scope of daily cleaning, clarification and care. No improvements can be expected from today to tomorrow. It can take several weeks before a noticeable difference occurs in a treatment, so consistency, patience and tenacity are required. Post-inflammatory hyperpigmentation may require medical treatment. If the size, shape or color of dark spots is a concern, it is a good idea to contact a dermatologist or pharmacist.

“Beauty is the greatest of human powers”!

Learn more with purebeau